NAVIGATOR *Evo*

SYSTEM FOR PREPARATION
OF ROOT CANALS

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Lengths 21; 25; 31 mm / Délky 21; 25; 31 mm

Design

Brand new Navigator EVO is the successor to the Wizard Navigator, the previous successful system. We used the best of Wizard Navigator and added new findings of our development team. The most noticeable change is material – thermal treated NiTi.

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Thermal treated NiTi

- + Greater flexibility
- + Higher cyclic fatigue resistance

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

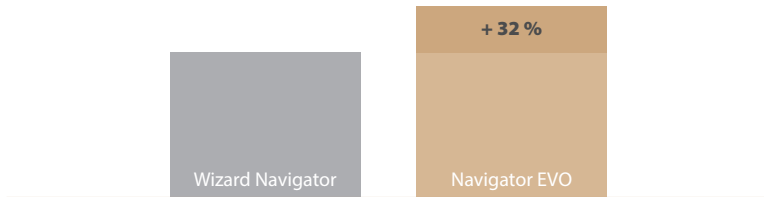
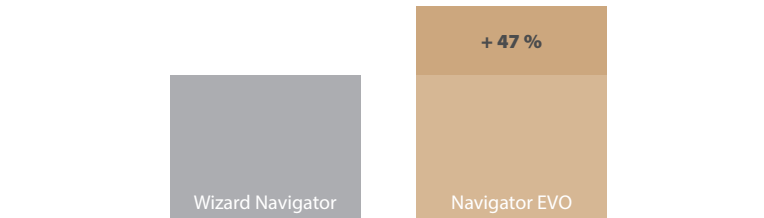
Working part design

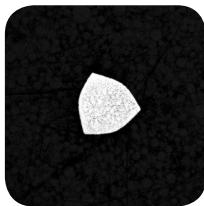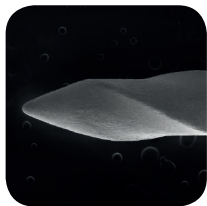Benefits

- + Can be used repeatedly
- + It is sterilizable
- + Efficient dentin removal
- + Inactive tip for easier guiding in root canals

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- « Inactive tip / Neaktivní špička
- « Instrument profile 3 mm from the tip / Profil nástroje 3 mm od špičky
- « Instrument profile 9 mm from the tip / Profil nástroje 9 mm od špičky

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Protocol

Navigator EVO instruments are designed for machine-assisted preparation using continuous clockwise motion with torque control.

Instructions for use

1. Primary clearing of a root canal using W-File 006, 008 or 010.
2. Measuring the working length – Apexlocator, X-ray.
3. Making the root canal passable with a hand instrument to the size of 015–020.
4. Widening the root canal using Navigator EVO W-1 or Navigator EVO W-2. The alteration of the instruments can be successfully used in highly complicated canals.
5. If necessary, the entrance to the root canal can be widened or moved using the W-XN instrument.
6. Consequently, peck the entire root canal using Navigator EVO W-3 to the dimension of 020.
7. Preparation using Navigator EVO W-4 with the possibility of filling.
8. Wide canals can be enlarged up to 035 using Navigator EVO W-6 with 4% or Navigator EVO W-8 with 6% taper and further to the size of 040 using Navigator EVO W-7 with 4% or Navigator EVO W-9 with 6% taper.

During preparations by individual instruments use 2–3 filing movements. | Rinse the root canals during the preparation frequently. | Set the endo-motor parameters (revolutions and twist) to the values according to the table. | For anatomically complicated canals, always check the passage of the canal. | Procedure of preparation – individual steps can be repeated. | The number of tools can be reduced in simple canals with good passability. | Place the used instruments to a cleaning box. | Record the number of uses of individual instruments in relation to the complexity of the root canal. | Navigator EVO instruments can be combined with the Unicone system. | We recommend using the products of MEDIN, a.s. for filling.

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"The system has improved, and the 10/04 instrument is now probably one of the most flexible instruments out there. Give it a try!"

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Bogdan M. Moldoveanu DMD, MDSc
Cluj-Napoca, Romania

"I used to be a big fan of the Wizard Navigator Endodontic System, but since MEDIN introduced Navigator EVO it is my system of the choice. Navigator EVO is really universal. It works efficiently in straight, curved, wide, narrow and other types of root canals. With those files endodontic treatment is fast, intuitive and what is very important to me – this system gives me good feedback during preparation."

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Michal Ganowicz DDS PhD
Warszaw, Poland

"I am working with MEDIN instruments. Wizard Navigator and Unicore are safe systems with great flexibility and sharpness. Navigator EVO is made of thermal-treated NiTi which improves flexibility and durability of the instruments."

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Prof. Dr. Gilson Blitzkow Sydney

Department of Endodontics – Universidade Federal do Paraná, Brasil

"MEDIN NiTi instruments philosophy is very familiar to me. The most important benefits are safeness and durability. Simplicity and 'friendly behaviour' during preparation are very pleasant too. Navigator EVO moves dentists opportunities forward again."

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